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EN Safety valve NO PVC M

Catalogue No.: V696433-M
Functionality: the manufacturer declares the functionality of the device when used in accordance with the instructions for use
Duration of use: „Short term“ - continuous use for the time maximum is 24 hours
Design: sterile, ethylene oxide sterilization
Target patient: Patients undergoing infusion therapy.
Designated user: Doctors, anesthetists, nurses, and other healthcare professionals in healthcare settings who perform infusion therapy.
Materials used: ABS, POLYISOPRENE
Compatibility: With medical devices equipped with standard Female Luer Lock connector in conformity with standard ČSN EN ISO 80369-7. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see www.sukl.cz or www.olecich.cz
Contraindication: Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects: No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.
Warning: Do not use, if the sterile packing is opened or damaged. Do not use after the expiry date.
 Reuse, re-sterilization, or non-observance of storage conditions creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage and transport: The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. Organic solvents and chemicals must not be stored with the products. If the package is exposed to conditions outside the specified conditions, contact the manufacturer who will decide on the next course of action.
 The products are transported in clean, dry and covered means of transport at a temperature from -15°C to +60°C and a relative humidity of max. 75% so that they are not damaged. No chemicals may be transported with them. The maximum transport time of MD at temperatures of +50°C to +60°C must not exceed 4 days!

Intended use

Single-use medical device intended for intermittent injection of drugs through a rubber latex-free stopper that allows repeated injection and protects against bacterial contamination. Safety valves can be used for both gravity and pressure infusion systems up to a pressure of 2 bar. The connecting part of the safety cap is compatible with Female Luer Lock cones that meet the requirements of the ČSN EN ISO 80369-7 standard.

Instruction for use

- Under aseptic conditions remove the safety valve from the packaging.
- Before application wipe or spray the surface of the rubber stopper with an alcoholic disinfectant.
- Wait for no less than 30 seconds until the disinfectant solution dries.
- Attach the valve with a clockwise twisting motion so as to ensure a safe connection.
- Detach the valve with a counter clockwise twisting motion.
- After use, products should be disposed of in line with prescribed procedures for disposal of medical (infectious) waste in accordance with local regulations.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number		Keep away from Rain	LOT	Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide and single sterile barrier system
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain PVC
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Pressure
MD	Medical Device		Date and Country of Manufacture	UDI	Unique Device Identifier		

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