Cévka pro novorozence CN-01 100 cm M

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Catheter for neonates CN-01 100 cm M

CE 1023

Catalogue No.:	V686417-01M
Functionality:	the manufacturer declares the functionality of the device when used in accordance with the instructions for use
Duration of use:	„Short term“ - continuous use for the time maximum is 24 hours
Design:	sterile, ethylene oxide sterilization
Target patient:	Supplemented - breastfed newborn.
Designated user:	Neonatologists, nurses and other healthcare workers in healthcare facilities who have relevant education and experience in caring for newborns.
Materials used:	PVC
Compatibility:	With Luer single-use syringe, Luer Lock single-use syringe (see Manufacturer's Note). CN-01 100 cm is not intended for administration of drugs.
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects:	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.
Warning:	While feeding a neonate from a feeding bottle, the bottle should be in level with the neonate's head. Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Reuse, re-sterilization, or non-observance of storage conditions creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage and transport:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. Organic solvents and chemicals must not be stored with the products. If the package is exposed to conditions outside the specified conditions, contact the manufacturer who will decide on the next course of action. The products are transported in clean, dry and covered means of transport at a temperature from -15°C to +60°C and a relative humidity of max. 75% so that they are not damaged. No chemicals may be transported with them. The maximum transport time of MD at temperatures of +50°C to +60°C must not exceed 4 days!

Intended use

A single use medical device that is used to feed a newborn who is breastfed but temporarily receives less milk from the breast than he needs. It allows the newborn to suck breast milk from the mother's breast and at the same time feed from a baby bottle or syringe.

Instruction for use

- Take the catheter out of the packaging under aseptic condition.
- The supplementary food can be supplied from a feeding bottle or a syringe:
 - feeding from a feeding bottle
 - cut the cap (cover) of the catheter connecting beam off
 - adapt the feeding bottle teat – cut it to create an opening allowing the adapted end of catheter to be pulled through (= the catheter connecting beam with a cut-off cap)
 - fill the feeding bottle with appropriate supplementary food recommended by the physician, and attach the teat to the bottle
 - pull the adapted catheter end through the opening in the teat, and dip it in the food
 - feeding from a syringe
 - open the cap (cover) of the catheter connecting beam
 - connect the syringe with supplementary food to the catheter connecting beam
- Position the neonate at the breast, and insert the rounded end of the catheter to the neonate's mouth so that he/she accepts both the papilla and the hose. This method perfectly maintains proper way of sucking; mother's hypophysis is stimulated; hormone prolactin is created, and, as a result, milk is produced.

Manufacturer's note: The manufacturer warns the user of the existing possibility of incorrect connection. The connection connector used on the catheter is a separately developed alternative connector. Compatibility with syringe has been verified by the manufacturer in physical and functional tests.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide and single sterile barrier system
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain DEHP phthalate
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Medical Device
	Date and Country of Manufacture		Unique Device Identifier				

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