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**EN Catheter for neonates of low birth weight CN-01**

**Catalogue No.:** V646959-ND  
**Product type:** medical device  
**Design:** sterile, ethylene oxide sterilization  
**Target patient:** neonate  
**Designated user:** medical personnel with appropriate education, knowledge and practical experience in medical devices.  
**Materials used:** PVC, PP  
**Compatibility:** Infusion sets and syringes must be equipped with Luer connection cone in conformity with standard ČSN EN ISO 80369-7. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see [www.sukl.cz](http://www.sukl.cz) or [www.olecich.cz](http://www.olecich.cz)  
**Contraindication:** Do not use on any patient with a known allergic reaction to any of the product components.  
**Adverse effects:** No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.  
**Warning:** The catheter shall not be used to evacuate the air passages.  
Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date.  
Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.  
**Storage:** The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation.  
Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

Intended use	
1.	Enteral nutrition fed to infants
2.	Medication to the umbilical vein applied when resuscitating the infant
3.	Short-time infusion to the umbilical vein

**Instruction for use**

ad. 1 Enteral nutrition fed to infants:  
Take the catheter out of the packing and introduce it the rounded tip first to the infant's stomach through its mouth or nose. The depth of catheter penetration can be checked using the graduation marks of 5, 10, 15 and 20 cm as measured from the rounded tip. Before the nutritious solution can be administered, make sure the catheter did not enter the lower air passages. Then attach the catheter via the connecting cone to a syringe or an infusion set and start delivering the solution.

To items 2 & 3 Medication to the umbilical vein  
Short-time infusions to the umbilical vein:  
Take the catheter out of the packing; open the cone closure and attach the cone to a syringe with infusion solution; fill the catheter with the solution. The rounded tip first introduce the catheter to the patient's umbilical vein. The depth of catheter penetration can be checked using the graduation marks of 5, 6 and 7 cm as measured from the rounded tip. Apply a short suction or relief action to make sure the catheter end is being filled with blood. Then re-attach the catheter to the syringe, the infusion set, etc., as the job at hand requires.

**PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:**  
Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

**NOTE TO USERS:** Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging							
	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain DEHP phthalate
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Medical Device
	Date of Manufacture						

The instruction for use is also available in electronic form at [www.gama.cz](http://www.gama.cz). Free copies of other copies are available on request via e-mail: [sales@gama.cz](mailto:sales@gama.cz).

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