CS

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**EN Nasal oxygen introduction catheter CK-472 M**

**Catalogue No.:** V646956-M  
**Functionality:** the manufacturer declares the functionality of the device when used in accordance with the instructions for use  
**Duration of use:** „Short term“ - maximum period of use 30 days. With continuous use of the tube 24 hours a day, replacement is necessary once a week, with sporadic intensity of use, replacement is possible every 2-4 weeks. Immediate replacement must be done if the coil becomes dirty, discolored or hardened.  
**Design:** sterile, ethylene oxide sterilization  
**Target patient:** Adults and children aged 15 and over undergoing oxygen therapy.  
**Designated user:** Doctors, nurses and other healthcare workers in healthcare facilities who are qualified to administer oxygen therapy. A lay person in a domestic environment.  
**Materials used:** PVC, PP  
**Compatibility:** With the outlet of the reducing valve of an oxygen pressure cylinder, an oxygenator or a central oxygen distribution using a funnel-shaped color-coded Charrière end. With gradual (step) connector of oxygen appliance. CK-472 is not intended for drug delivery.  
**Contraindication:** Do not use on any patient with a known allergic reaction to any of the product components.  
**Adverse effects:** No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.  
**Warning:** Do not use if the sterile packing is opened or damaged. Do not use after the expiry date.  
 Reuse, re-sterilization, or non-observance of storage conditions creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.  
**Storage and transport:** The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation.  
 Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. Organic solvents and chemicals must not be stored with the products. If the package is exposed to conditions outside the specified conditions, contact the manufacturer who will decide on the next course of action.  
 The products are transported in clean, dry and covered means of transport at a temperature from -15°C to +60°C and a relative humidity of max. 75% so that they are not damaged. No chemicals may be transported with them. The maximum transport time of MD at temperatures of +50°C to +60°C must not exceed 4 days!

**Intended use**

A single-use medical device intended for the introduction of oxygen in the therapy of adults and children from 15 years of age.

**Instruction for use**

1. Check the type labelling of the catheter, usable life on the package and the package for damages.
2. Take the catheter out of the packaging under aseptic condition.
3. Insert nasal ducts into nasal openings.
4. Put white sleeve pieces on ears (like glasses).
5. Connect the ending to the breathing apparatus.
6. The catheter is ready for oxygen treatment.

**PROCEDURE FOR THE SAFE DISPOSAL OF MD AFTER USE:**

**After using MD in a medical facility:** Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

**After using the MD in the home environment:** rinse the MD with warm soapy water, wrap it in a plastic bag and put it in a trash can intended for municipal waste.

**NOTE TO USERS:** Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

**CIRCUMSTANCES WHICH THE USER SHOULD CONSULT with A MEDICAL PROFESSIONAL:** If difficulties are experienced during use (convulsions, lung irritation, etc.), contact your doctor.

**Used symbols on packaging**

	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide and single sterile barrier
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain DEHP phthalate
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Medical Device
	Date and Country of Manufacture		Unique Device Identifier				

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