CS **GAMAPLUS**

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EN **GAMAPLUS**

Catalogue No.: V606301-ND V606301-01ND V606303-ND V606304-ND V606304-01ND	Medical device variants: GAMAPLUS 1,8 x 450 LL GAMAPLUS 1,8 x 450 LLK GAMAPLUS 1,8 x 1500 LL GAMAPLUS 1,8 x 1800 LL GAMAPLUS 1,8 x 1800 LLK	Catalogue No.: V606306-ND V606307-ND V606321-ND V606342-ND	Medical device variants: GAMAPLUS 1,8 x 450 UNIV GAMAPLUS 1,8 x 1800 UNIV GAMAPLUS 3,0 x 150 LL GAMAPLUS 3,0 x 300 LL	Catalogue No.: V606345-ND V606347-ND V606349-ND V606371-ND	Medical device variants: GAMAPLUS POCARD GAMAPLUS 3,0 x 2500 LL GAMAPLUS 3,0 x 3500 LL GAMAPLUS 3,0 x 150 YLL
Product type:	medical device				
Design:	sterile, ethylene oxide sterilization				
Target patient:	population of pediatric or adult patients				
Designated user:	medical personnel with appropriate education, knowledge and practical experience in medical devices.				
Materials used:	PVC, PE				
Compatibility:	With medical devices equipped with standard Luer connectors in conformity with standards ČSN EN ISO 80369-7. GAMAPLUS is designed to transfer liquid or i.v. administration or transfer of blood. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see www.sukl.cz or www.olecich.cz				
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.				
Adverse effects:	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.				
Warning:	Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.				
Storage:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.				

Intended use

GAMAPLUS hose is intended for interconnection of one or two infusion or transfusion kits with the intravenous needle.

Instruction for use

1. Take GAMAPLUS out of the packing.
2. Remove the covers.
3. Attach the negative (female) cone to the positive (male) cone of the set and fill the hose with liquid.
4. Connect the positive cone to the intravenous needle or a catheter.

Procedure for safe disposal of medical devices after use:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number	Keep away from Rain	LOT	Batch Number	Manufacturer
Use by / Expiry date		Temperature Limitation	Read instructions for use		Sterilised by Ethylene Oxide
Pyrogen free		Keep away from sunlight	Latex free product		Does not contain DEHP phthalate
Do not Resterilize		For single use only	Do not use if unit package is damaged		Pressure
VOL	Volume of tubing	MD		Date of Manufacture	

The instruction for use is also available in electronic form at www.gama.cz. Free copies of other copies are available on request via e-mail: sales@gama.cz

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