GAMAPLUS 3,0 x 450 MVT

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GAMAPLUS 3,0 x 450 MVT

Catalogue No.:	V606351-ND
Product type:	medical device
Design:	sterile, ethylene oxide sterilization
Target patient:	population of pediatric or adult patients
Designated user:	medical personnel with appropriate education, knowledge and practical experience in medical devices.
Materials used:	PVC, PE, PP, PTFE membrane 0,2 µm
Compatibility:	With medical devices equipped with standard Luer connectors in conformity with standards ČSN EN ISO 80369-7. GAMAPLUS is designed to transfer liquid or i.v. administration or transfer of blood. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see www.sukl.cz or www.olecich.cz
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects:	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.
Warning:	Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

Intended use

GAMAPLUS 3x450 MVT is employed to perform dialysis using the GAMBRO, FRESENIUS, ALTHIN & HOSPAL monitors in case the situation requires that the number of pressure gages to be attached exceeds the number of gages that the dialyzing set is designed for. The attachment is achieved by screwing the positive threaded cone to some of the manifold branches of the dialyzing sets. GAMAPLUS 3x450 MVT is not a part of the dialyzing set; it is disposable, intended for a single use.

Instruction for use

1. Check the type designation on the set packing, inspect the packing if undamaged and make sure no cover of the set dropped off. Do not use the set if there is a discrepancy between the set and its type designation or a in case the packing is damaged, or when the protective covers dropped off.
2. Take the item out of the packing.
3. Remove the cover from the positive cone on GAMAPLUS 3x450 MVT.
4. Screw the positive cone on an available, i.e. free, manifold branch of the dialyzing set.
5. Break off the flexible part of the separating filter and screw on the pressure gage.
6. Open the pinchcock on the used branch of the dialyzing set.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number	Keep away from Rain	LOT	Batch Number	Manufacturer
Use by / Expiry date		Temperature Limitation	Read instructions for use		Sterilised by Ethylene Oxide
Pyrogen free		Keep away from sunlight	Latex free product		Does not contain DEHP phthalate
Do not Resterilize		For single use only	Do not use if unit package is damaged		Pressure
MD	Medical Device	Date of Manufacture	VOL	Volume of tubing	

The instruction for use is also available in electronic form at www.gama.cz. Free copies of other copies are available on request via e-mail: sales@gama.cz.

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