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EN **HD Sampling set**

Catalogue No.: V606137-ND
Product type: medical device
Design: sterile, ethylene oxide sterilization
Target patient: it is not intended for patients
Designated user: medical personnel with appropriate education, knowledge and practical experience in medical devices.
Materials used: PC, PE, PVC
Compatibility: With dialysis monitors in conformity with standard ČSN EN 60601-2-16. The diameter of the holder and filter must be 47 mm and the holder and filter must be produced by Millipore or Sartorius. Blood pumps with the fixed dimension of pump segment tubes of 8.0 x 12.0 mm. HD Sampling set is not intended for drug delivery.
Warning: Do not use, if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Repeated use or repeated sterilization creates it may lead to contamination or impairment of set functional.
Storage: The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

Intended use

HD Sampling Set serves for taking a bacteriological sample from saline solution of a hemodialysis device before the beginning of the treatment procedure or after its end.

Instruction for use

1. Take out the set from the packaging.
 2. At setting a dialysis monitor follow the operating manual of the monitor manufacturer.
 3. Install a pump segment into a blood pump.
 4. Connect a positive cone to a filter holder.
 5. Connect a three-way cock to an infusion port of the dialysis monitor.
 6. After opening the three-way cock sample the required amount of saline solution.
 7. After sampling turn the three-way cock in the way so as to let only air into the filter.
 8. After pumping of saline solution from the set, close spring pinch-cocks on the inlet and outlet of the filter holder.
 9. Behind the closed pinch-cocks, separate the HD Sampling Set from the filter holder.
 10. Hand over the separated filter holder closed with pinch-cocks including a filter bed for microbiological cultivation.
- ⚠ Before sampling make sure that all joints are tightened, close-fitting and all connectors are safely connected. During installation of the HD Sampling Set observe all principles of aseptic work.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number	Keep away from Rain	LOT	Batch Number	Manufacturer
	Use by / Expiry date	Temperature Limitation		Read instructions for use	Sterilised by Ethylene Oxide
	Pyrogen free	Keep away from sunlight		Latex free product	Does not contain DEHP phthalate
	Do not Resterilize			Do not use if unit package is damaged	Attention
MD	Medical Device			Date of Manufacture	

The instruction for use is also available in electronic form at www.gama.cz. Free copies of other copies are available on request via e-mail: sales@gama.cz.

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