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## EN NASAL ASPIRATOR – BLUE / NASAL ASPIRATOR - PINK

**Intended use:** The nasal aspirator is intended for children under 3 years who don't know how to clean their noses. It is used for removal of excessive phlegm from infant's nasal cavity that may result in cough or inflammation of the middle ear. The benefit of the aspirator lies in continuous control and check of the suction.

**Description:** lid, oral end piece, nasal end piece – polyvinylchloride (PVC) without phthalates, chamber – polystyrene (PS). Transparent design allows user to check the sucked aspirator volume.

**Safety components:** rounded and structural shape of the nasal end piece prevents from injury of child's mucosa.

**Cleaning:** after unpacking disassemble the aspirator by removing of the lid from the chamber. Wash all components in warm water (max 60°C) with baby soap or detergent intended for washing of feeding bottles. Thoroughly wash and dry all components. Repeat disinfection (washing) before next use.

### Instruction for use:

1. Disinfect the aspirator according to "Cleaning" clause.
2. Put your baby in comfortable supine position (or let your baby to sit on your knees), turn baby's face to you and hold the baby's head to restrain movement during cleaning of the nose.
3. Attach the nasal end piece to baby's nasal cavity.
4. Put the oral end piece to your mouth and suck air. When sucking air through the end piece, pressure is created to remove phlegm from the nose. Control the suction force as needed.

*Phlegm is often too thick or dry and cannot be removed. In these cases administration of physiological NaCl solution or seawater solution drops to the nostrils is recommended.*

**Warning:** The product must not be sterilized by boiling water, steam, or hot air! Check the quality of the product before each use. In case of first symptoms of damage or wear and tear – discard the product.

If your baby has some injury inside the nose, consult use of the aspirator with physician.

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