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www.gama.cz

Revízie č. / Revízia č. 2, 5.10.2022.

## EN Nasal aspirator with Vacuum Cleaner Adapter

### ASSEMBLY, EXTRACTING, CLEANING

1. After removing the nasal aspirator from the package, rinse the mucus extractor in hot soapy water and rinse thoroughly with warm water (max 60 °C). 2. After washing, close the cup properly with the lid with hoses. 3. Connect the hose end of the mucus extractor with the attachment cone adapter. 4. Start the vacuum cleaner as instructed and insert the attachment cone adapter of the nasal aspirator into the suction hose. 5. Put the nasal tip to the baby's nasal cavity and extract the aspirator into the cup. If necessary, repeat similarly with the other nostril. A higher suction effect can be achieved by covering the free nostril. 6. When the suction has finished, turn off the vacuum cleaner and disconnect the hose cone adapter from the vacuum cleaner. Remove the attachment cone adapter from the vacuum cleaner. Take the mucus extractor apart, remove the lid from the cup and clean it according to point 1.

### WARNING

1. Use a standard household vacuum cleaner with power of 800 W - 1800 W with suction power control. Reduce the suction power to a minimum before using the vacuum cleaner!
2. Do not use the nasal aspirator for any purpose other than that specified by the manufacturer.
3. Do not use the nasal aspirator if any of the components are missing or damaged.
4. Do not expose the nasal aspirator to extreme temperatures and protect it from direct sunlight.
5. The nasal aspirator cannot be sterilized by boiling, steam, or hot air.

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### EFFECTIVITY

it also removes the "back rhinitis" from the nasal cavities and nasopharynx»it reduces the risk of transferring the infection between the parent and the child

### SAFETY

it contains no glass parts»it is made of health-conscious materials»it is non-toxic»it is equipped with safety nose end piece»it is certified as a medical device

### PRACTICALITY

usable also without a vacuum cleaner»it can be disassembled and it's easy to clean»all the mucus is extracted at once and thoroughly»all the work is carried out by a vacuum cleaner

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