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(EN) **Separation filter**

Catalogue No.: V606991-ND
Product type: medical device
Design: sterile, ethylene oxide sterilization
Target patient: it is not intended for patients
Designated user: medical personnel with appropriate education, knowledge and practical experience in medical devices.
Materials used: PVC
Compatibility: With medical devices equipped with standard Luer Lock connectors in conformity with standard ČSN EN ISO 80369-7.
Warning: Do not use, if the sterile packing is opened or damaged. Do not use after the expiry date. Repeated use or repeated sterilization creates it may lead to contamination or impairment of set functional.
Storage: The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

Intended use

The Separation filter is designed to protect the pressure transducers of medical devices from ingress of blood or other liquid.

Instruction for use

1. Under aseptic conditions remove the filter from the packaging.
2. Attach the filter with a clockwise twisting motion so as to ensure a safe connection.
3. Detach the filter with a counter clockwise twisting motion.

Procedure for safe disposal of medical devices after use:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number	Keep away from Rain	LOT	Batch Number	Manufacturer
Use by / Expiry date		Temperature Limitation	Read instructions for use	Sterilised by Ethylene Oxide	
Pyrogen free		Keep away from sunlight	Latex free product	Does not contain DEHP phthalate	
Do not Resterilize		For single use only	Do not use if unit package is damaged	Medical Device	
Date of Manufacture					

The instruction for use is also available in electronic form at www.gama.cz. Free copies of other copies are available on request via e-mail: sales@gama.cz.

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