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(EN) **Transfer set M**

Catalogue No.:	V606386-M
Functionality:	the manufacturer declares the functionality of the device when used in accordance with the instructions for use
Duration of use:	„Transient“ - continuous use for a maximum of 60 minute
Design:	sterile, ethylene oxide sterilization
Target patient:	Cataract patients.
Designated user:	An ophthalmologist, also known as an ophthalmologist or specialist in ophthalmology.
Materials used:	PE, PVC
Compatibility:	With microsurgical device and MILLENNIUM handpiece. The transfer set is not intended for drug delivery.
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects:	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.
Warning:	Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Reuse, re-sterilization, or non-observance of storage conditions creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage and transport:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. Organic solvents and chemicals must not be stored with the products. If the package is exposed to conditions outside the specified conditions, contact the manufacturer who will decide on the next course of action. The products are transported in clean, dry and covered means of transport at a temperature from -15°C to +60°C and a relative humidity of max. 75% so that they are not damaged. No chemicals may be transported with them. The maximum transport time of MD at temperatures of +50°C to +60°C must not exceed 4 days!

Intended use

A single-use medical device designed to transfer the infusion solution during eye surgeries.

Instruction for use

- Under aseptic conditions, remove the set from the packing.
- After removing a cover, connect the female Luer lock (marked with a red cover) of the aspiration tubing to the aspiration connector of the working adapter.
- Remove the remaining Luer lock covers.
- Connect the male Luer lock of the irrigation tubing to the irrigation connector of the working adapter.
- Connect the other Luer locks on the opposite aspiration and irrigation tubing ends to the corresponding case connectors.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

REF	Catalogue Number	Umbrella	Keep away from Rain	LOT	Batch Number	Factory	Manufacturer
Hourglass	Use by / Expiry date	Thermometer	Temperature Limitation	Book	Read instructions for use	STERILE EO	Sterilised by Ethylene Oxide and single sterile barrier system
Pyrogen free	Pyrogen free	Sun	Keep away from sunlight	LATEX	Latex free product	DEHP	Does not contain DEHP phthalate
Do not Resterilize	Do not Resterilize	Single use only	For single use only	Do not use if unit package is damaged	Do not use if unit package is damaged	MD	Medical Device
Date and Country of Manufacture	Date and Country of Manufacture	UDI	Unique Device Identifier				

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