CS

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CE 1014

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EN

Transfer set PS

Catalogue No.:	V606386-02ND V606386-03ND
Product type:	medical device
Design:	sterile, ethylene oxide sterilization
Target patient:	population of pediatric or adult patients
Designated user:	medical personnel with appropriate education, knowledge and practical experience in medical devices.
Materials used:	PE, ABS, PVC
Compatibility:	The transfer set is compatible with Handpiece Bausch & Lomb. The transfer set PS is not intended for drug delivery.
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects:	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.
Warning:	Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.

Intended use

The transfer set is designed to transfer infusion solutions during the eye operations by using a microsurgical instrument: Stellaris and Stellaris PC

Instruction for use

- Under aseptic conditions, remove the set from the packing.
- After removing the cover, connect the blue cone of aspiration tubing to the aspiration connector of Handpiece.
- Then remove covers of the remaining cones.
- Connect the green cone of irrigation tubing to the irrigation connector of Handpiece.
- Connect the cones at opposite ends of aspiration and irrigation tubing to the relevant connectors of the cartridge.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain DEHP phthalate
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Medical Device
	Date of Manufacture						

The instruction for use is also available in electronic form at www.gama.cz. Free copies of other copies are available on request via e-mail: sales@gama.cz.

 GAMA GROUP a.s., Betrieb Jimramov – 592 42 Jimramov
Sitz der Gesellschaft GAMA GROUP a.s., Mánesova 11/3b, České Budějovice 7, 370 01 České Budějovice, Tschechische Republik, Id.Nr.: 45797463, www.gama.cz

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