Rampa s bezjehlovými vstupy M

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Ramp with needleless ports M

Catalogue No.:	Medical device variants:
V696423-M V696425-M	Ramp 3-way with NP NO PVC M Ramp 5-way with NP NO PVC M
Functionality:	the manufacturer declares the functionality of the device when used in accordance with the instructions for use
Duration of use:	„Short term“ - continuous use for the time max. 7 days or 100 connections
Design:	sterile, ethylene oxide sterilization
Target patient:	Patients undergoing intravenous or intra-arterial infusion therapy where concomitant administration of infusions or drugs is required.
Designated user:	Physicians, anesthetists, nurses, and other healthcare professionals in healthcare facilities who administer intravenous or intra-arterial infusion therapy.
Materials used:	PC, PSU, PE, hydrophobic lubricant
Compatibility:	With medical devices equipped with standard Luer connectors under ČSN EN ISO 80369-7. With liquid drugs, infusion solutions or lipid emulsions. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see www.sukl.cz or www.olecich.cz
Contraindication:	Do not use on any patient with a known allergic reaction to any of the product components.
Adverse effects:	No adverse effects are known if the ramp is used in accordance with the instructions specified in the Warning and Use sections.
Warning:	Prior to use, check all the taps for operability. Use standard Luer connection only, which complies with harmonised standards, so as not to damage the needleless ports. When using the ramps it is necessary to adhere to the principles of antisepsis, sterility and the specified replacement period must be respected. Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Reuse, re-sterilization, or non-observance of storage conditions creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.
Storage and transport:	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. Organic solvents and chemicals must not be stored with the products. If the package is exposed to conditions outside the specified conditions, contact the manufacturer who will decide on the next course of action. The products are transported in clean, dry and covered means of transport at a temperature from -15°C to +60°C and a relative humidity of max. 75% so that they are not damaged. No chemicals may be transported with them. The maximum transport time of MD at temperatures of +50°C to +60°C must not exceed 4 days!

Intended use
 The needleless port ramps (hereafter referred to as the ramp) is intended for simultaneous infusion therapy during infusion or drug administration. The ramp is intended for connection to a central venous catheter and for pressure and gravity applications up to a pressure of 1.5 bar. It allows setting, controlling, controlling or stopping the desired direction of fluid flow. The connection cones of the needle-free inlets are compatible with Luer cones. The ramp must always be replaced after a maximum of 7 days or after 100 connections (one needle-free input = 100 connections).

- Instruction for use**
- Under aseptic conditions remove the ramp from the packaging and remove the guard from the connection cone.
 - Before application, wipe or spray the surface of the silicone diaphragm of needleless ports with an alcoholic disinfectant.
 - Wait for no less than 30 seconds until the disinfectant solution dries out.
 - Connect the ramp using the connection - extension tubing (not included in the kit) to a central venous catheter to ensure a safe connection.
 - Attach the needleless ports to infusion or injection lines with Luer ending so as to ensure a safe connection.
 - Turn the taps to set the desired direction of flow.

PROCEDURE FOR SAFE DISPOSAL OF MEDICAL DEVICES AFTER USE:
 Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging							
	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide and single sterile barrier system
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain PVC
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Pressure
	Medical Device		Date and Country of Manufacture		Unique Device Identifier		

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