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EN Urine bag

| | | | |
|--|--|--|---|
| Catalogue No.: V606704 V606705 V606709 | Medical device variants: Urine bag SU 20 V1 Urine bag SU 15 Urine bag SUP 20 | Catalogue No.: V606710 V616701 V616702 | Medical device variants: Urine bag SUP 20 V2 Urine bag SU 20 Urine bag SU 20 V2 |
| Product type: | medical device | | |
| Design: | non-sterile | | |
| Target patient: | population of pediatric or adult patients | | |
| Designated user: | medical personnel or home user with knowledge and practical experience of these medical devices. | | |
| Materials used: | PVC, PE, ABS | | |
| Compatibility: | With urological catheters terminated by tunnel-shaped connection Charrière. | | |
| Contraindication: | Do not use on any patient with a known allergic reaction to any of the product components. | | |
| Adverse effects: | No adverse effects are known if the bag is used in accordance with the instructions specified in the Warning and Use sections. | | |
| Storage: | The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%. | | |

Intended use

Urinal bags are designed for single as well as multiple collecting of catheterized patients' urine. The bags are adapted to be carried in hand or on the body, or to be hanged on a patient's bed. The variants without a discharge valve are designed for single filling; the variants with discharge valves are designed for multiple use with one patient. The bags are supplied as non-sterile.

Instruction for use

- Before the bag is put to use, make sure the drain outlet is closed (if any provided).
- Using the attachment strip hang the bag onto the bed frame or using hooks suspend the bag by the holes available at its top; fix the hose so that it cannot bend and constrict impassable.
- Take off the protective cover and attach the connecting cone to the catheter.
- Keep the urine bag under continuous surveillance and empty the bag if fitted with a drain outlet.

The scale on the bag offers only a rough indication; do not use it as a metering device.

Caution when emptying urine from a urinal bag with a spout:

- Wash your hands with soap and water.
- Hold the bag spout over the toilet or over a collecting vessel.
- Open the spout and leave the urine to drain out of the bag (do not touch the toilet or collecting vessel with the spout).
- Once the bag is empty, wipe the spout with a disinfectant containing alcohol, close the spout and wash your hands with soap and water.

NOTE: After use, products should be disposed of in line with prescribed procedures for disposal of medical (infectious) waste in accordance with local regulations.

NOTE TO USERS: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

Used symbols on packaging

| | | | | | | | |
|-----------|----------------------|-------------|-------------------------|-------|---------------------------|-------------|---------------------------------|
| REF | Catalogue Number | Umbrella | Keep away from Rain | LOT | Batch Number | Factory | Manufacturer |
| Hourglass | Use by / Expiry date | Thermometer | Temperature Limitation | Book | Read instructions for use | NON STERILE | Non-sterile |
| Warning | Attention | Sun | Keep away from sunlight | LATEX | Latex free product | PHT/DEHP | Does not contain DEHP phthalate |

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